

Link to the product: <https://sforne.com/bathmate-hydroxtreme-7-penis-pump-x30-p-4342.html>



## BATHMATE - HYDROXTREME 7 PENIS PUMP X30

Price	<b>300.65 £</b>
Number	<b>70137184</b>
Producer code	<b>FETISHID-205448</b>
EAN	<b>5060140209287</b>

### Product description

**Bathmate** is the latest and most innovative penis developer with hydraulic technology. Get powerful erections safely and easily, prolonging the duration and hardness while developing and enlarging your penis.

Thanks to the enormous potential of hydraulic force, Bathmate can exert greater suction power than its competitors, forcing the expansion of the corpora cavernosa and increasing your penis size in minimal time.

With Bathmate you can achieve a size increase discreetly and peacefully in the privacy of your bathtub or shower.

Advantages and features:

- Rotating duel position for maximum comfort.
- Designed for use with Hydromax pleasure rings.
- Ultra soft pad that ensures a solid seal.
- Incredible effect in the water and out of the water.
- Valve to allow the penis to be exercised better.

Thousands of men who have already enjoyed Bathmate have been pleasantly surprised by the quick results achieved. By using Bathmate we can get the extra length and thickness we have always wanted. Perhaps you didn't know that you could significantly increase the size of your penis, in the comfort and privacy of your own bathroom with amazing results, in just 15 to 20 minutes per day for 6 weeks.

---

With Bathmate you will get a much thicker, longer and healthier penis, giving you permanent gains, which can be enjoyed forever! Many men are using this device right now, sitting in their bathtub or shower...getting instant, effective, and fast-progressing results only with Bathmate.

How is it used?

- 1) Fill the bathtub with water.
- 2) U Once you are in the bathtub, place the Bathmate under water until it fills completely. The penis is then introduced inside and rests on the base of the pelvis.
- 3) Push the Bathmate towards you repeatedly. This operates the hydraulic contraction mechanism of the pump base, creating the vacuum effect. You will begin to feel the pressure increases. As water is evacuated from inside through the upper valve, the penis expands more and more.
- 4) Once you can't pump any more, simply rest for five minutes. The most optimal vacuum is when you can no longer expel any more water from inside the Bathmate.
- 5) Repeat the pumping operation every four or five minutes, rest and repeat the maneuver. It is necessary to repeat this operation, since in each compression cycle, there is a moment when no more water can be extracted.
- 6) Repeat the pumping operation every four or five minutes, rest and repeat the maneuver. It is necessary to repeat this operation, since in each compression cycle, there is a moment when no more water can be extracted.
- 7) If you feel the pressure is too strong, please release the top valve slightly. Remember to do it in water to prevent air from entering. After 15 to 20 minutes of use, reduce the pressure by pressing the pressure release on the top of the Bathmate. Then stand and see immediate results. You will not be disappointed!
- 8) It is recommended to use 15 to 20 minutes per session, three times a week to obtain better results. Start feeling the power of lengthening your penis in an efficient way!!

## Pump Guide

### Penis Pump Guide

Manual pumps offer more control. Automatic pumps are easier but less adjustable. Hydro pumps (Bathmate) use water for even pressure.

Start with low pressure and short sessions. Increase gradually over weeks. Never sleep with a pump on.