

Link to the product: <https://sforne.com/bathmate-hydroxtreme-9-penis-pump-x40-p-4343.html>



BATHMATE - HYDROXTREME 9 PENIS PUMP X40

Price	357.02 £
Number	70137185
Producer code	FETISHID-205449
EAN	5060140209249

Product description

Bathmate HYDROXTREME 9 is the latest and most innovative penis enhancer with hydraulic technology. Achieve powerful erections safely and easily, prolonging their duration and hardness while developing and enlarging your penis.

15% longer than the X7 Xtreme! Discover a higher level! Advantages and features:

- Rotating dual position for maximum comfort.
- Designed for use with Hydromax pleasure rings.
- Ultra-soft pad ensures a solid seal.
- Incredible effect in and out of the water.
- Valve to allow the penis to be exercised better.

Thousands of men who have already enjoyed Bathmate have been pleasantly surprised by the rapid results achieved. By using Bathmate, we can achieve the extra length and thickness we've always wanted.

Perhaps you didn't know that you could significantly increase the size of your penis, in the comfort and privacy of your own bathroom with amazing results, in just 15 to 20 minutes a day for 6 weeks.

With Bathmate, you'll get a much thicker, longer, and healthier penis, giving you permanent gains you can enjoy forever! Many men are using this device right now, sitting in their bathtub or shower...getting instant, effective, and rapidly progressing results only with Bathmate.

How do you use it?

- Fill the bathtub with water.
- Once you're in the tub, place the Bathmate under the water until it's completely full. Then insert your penis inside and rest it on the base of your pelvis.
- Push the Bathmate toward you repeatedly. This activates the hydraulic contraction mechanism at the base of the pump, creating a vacuum effect. You'll begin to feel the pressure increase. As the water is evacuated from the interior through the top valve, the penis expands more and more.
- Once you can't pump any more, simply rest for five minutes. The optimal vacuum is reached when you can no longer

expel any more water from inside the Bathmate.

- Repeat the pumping process every four or five minutes, rest, and then repeat. This process must be repeated because, in each compression cycle, there comes a point where no more water can be extracted.
- If you feel the pressure is too strong, gently release the top valve. Remember to do this while still in the water to prevent air from entering. After 15 or 20 minutes of use, reduce the pressure by pressing the pressure release button on the top of the Bathmate. Then stand up and experience the immediate results. You won't be disappointed!
- It is recommended to use it for 15 to 20 minutes per session, three times a week for best results. Start feeling the power of lengthening your penis efficiently!

Pump Guide

Penis Pump Guide

Manual pumps offer more control. Automatic pumps are easier but less adjustable. Hydro pumps (Bathmate) use water for even pressure.

Start with low pressure and short sessions. Increase gradually over weeks. Never sleep with a pump on.