

Link to the product: <https://sforne.com/intimichic-menstrual-cup-medical-grade-silicone-size-s-p-1658.html>



## INTIMICHIC - MENSTRUAL CUP MEDICAL GRADE SILICONE - SIZE S

Price	<b>45.41 Euro</b>
Number	<b>70134500</b>
Producer code	<b>FETISHID-213038</b>
EAN	<b>8425402155240</b>

### Product description

Intimichic Cup has been developed with specialized doctors. It is the most innovative and healthy alternative to the use of tampons and disposable pads. Intimichic Cup is a reusable menstrual cup that adapts perfectly to the vaginal walls, collecting menstrual flow, instead of absorbing it, without leaks, without altering the vaginal mucosa and without impeding natural lubrication.

- Made of 100% medical grade silicone, the safest material in contact with biocompatible skin that does not cause any type of allergic reaction or inflammation.
- Compatible with the IUD and the contraceptive ring.
- Perfect for using the cup while doing sports, traveling or sleeping.

#### How to choose the right size?

- If you are under 25 years old, use size S, if you are under 25 years old but have had a VAGINAL birth, (without cesarean section) use size L.
- If you are over 25 years old, use size L, if you are over 25 years old but have had a cesarean birth, use size S.

The INTIMICHIC menstrual cup is also the most economical alternative to using other disposable products, since its resistance allows it to be used for approximately 10 years. Its cost is amortized over 6 months.

#### How to use it?

Intimichic is very easy to use and place, the most important thing is to be relaxed and find the most comfortable position for both insertion and extraction.

#### The steps to follow are;

1. Have your hands completely clean
2. Sterilize INTIMICHIC CUP for 3 minutes in boiling water.
3. Apply INTIMICHIC CLEANER sterilizer and let the cup dry.
4. Apply a little water-based lubricant before the first introduction, it will make insertion easier.
5. It is advisable to find the most comfortable position for easy insertion.
6. Hold the cup with your fingers and compress it in the lowest possible position and insert it obliquely into the vagina towards the bottom, the placement of a vaginal cup is lower than a tampon.
7. After insertion into the vagina, release it and let it unfold, it will automatically adapt to the vaginal wall.

Removal is very simple, pull the lower sleeve and automatically carefully remove it to empty the contents.

Once its use has finished.

- 
1. Clean the menstrual cup with hot water, pH neutral soap and finish by cleaning it with the INTIMICHIC sterilizer.
  2. Store INTIMCHIC CUP in the bag included inside the box.

In each container you will find

- INTIMICHIC medical silicone grade cup
- Cloth bag
- Instructions for use.

INTIMICHIC CUP is the best developed menstrual cup in the world thanks to the collaboration of medical specialists and obtaining certificates for the manufacturing material.

- Clinically tested and 100% hypoallergenic
- Long useful life
- Soft and smooth interior texture.
- Biocompatible
- Thanks to its material, cleaning is very simple and completely free of bacteria.